BT91

Self-Contained Biological Indicator

For plasma or vaporized Hydrogen Peroxide sterilization processes.

Usage

Monitoring plasma or vaporized Hydrogen Peroxide (H₂O₂) sterilization processes.

Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016. ISO 11138-1:2017 and IRAM 37102-1:1999.

Classification

Class 1, according to risk.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Characteristics

Polypropylene tube: 45.2 mm high x 8.5 mm external diameter. Wall thickness: 0.5 mm.

Polypropylene cap: 13.0 mm high x 10.9 mm external diameter. Wall thickness: 0.7 mm.

Cap filter: polyethylene fibers, 17.0 mm diameter, 64.5 g/m².

Glass ampoule: 35.0 - 40.0 mm high. External diameter: 6.5 mm. Wall thickness: 0.2 - 0.3 mm.

Culture medium: 0.5 - 0.7 ml, **purple** color.

10⁵ or 10⁶ *Geobacillus stearothermophilus* ATCC 7953 spores per vial.

Upon completion of sterilization process, incubate the indicator for a maximum of 24 hours at temperature between 55-62 °C.

Positive growth results (failure in the sterilization process): culture medium turns to **yellow** after incubation between 55-62 °C for 24 hours (maximum).

D value at 50 °C, 2.0 mg/l H₂O₂.

Environmental conditions during manufacture

T= 15-30 °C, RH= 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

Storage conditions

T= 10-30 °C, RH= 30-80 %, keep in a dark place in its original box.

Transport conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages.

The transport of this product does not represent any risk for human health.

Shelf life

2 years.

Packing

100 units per box.

Packing information: product code and description, process for intended use, presentation, regulation, bacterial strain, storage conditions, manufacturer information and data on box's label.

Labelling

On the product: 17.0 mm x 33.0 mm polypropylene label printed in black. 1.5 mm chemical indicator line, printed with Hydrogen Peroxide reactive ink (color change to green). Product code, batch number, expiration date, process for intended use and name of organism.

On product's packing: product code, batch number, bacterial load, manufacture and expiration date, barcode and datamatrix code.

NOTE: manufacture date is calculated by subtracting 24 months to the expiration date.

Possible target markets

Healthcare and Industry.

Other important information

It is advisable to incubate at 60 °C in Bionova® IC10/20 incubator.

Read product's directions for use thoroughly before use.

Precautions

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat, Radiation or any sterilization process other than Hydrogen Peroxide.

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